

JONASET BREATHING AIR WARMER

Jonaset breathing air warmer helps you stay active in cold weather. The device contains a stainless steel mesh core that captures the heat of exhaled air and uses it to warm the inhaled air. At the same time, it humidifies the dry winter air to a more comfortable level for the body.

INSTRUCTIONS FOR USE:

1. Attach the Jonaset warmer inside a neck warmer, hood, or scarf. Depending on the weather and intended use, you can choose the thinner white model or the more efficient blue model.
2. Position the warmer carefully in front of your mouth and nose, ensuring it does not cover your eyes. You may gently bend the steel mesh to better fit your facial contours or trim the edges with scissors if needed. Do not cover the steel mesh with anything other than the manufacturer's knit materials, which have sufficiently low breathing resistance.
3. Eyeglass wearers can bend the upper metal clip to help prevent lens fogging. If fogging still occurs—especially while driving or in other potentially hazardous situations—discontinue use immediately.
4. This product is intended only for warming inhaled air and does not filter fine particles.
5. The product is for personal use only. After each use, rinse the warmer under running water. For deeper cleaning, use a liquid detergent 2–3 times per week depending on frequency of use.
Note: The soft edges of the warmer must not be exposed to temperatures above 60 °C.
6. The warmer is made of skin-friendly polyethylene foam and acid-resistant stainless steel mesh. Dispose of the product as mixed waste; metal parts should go in metal recycling.