

JONASPORT BREATHING AIR WARMER

Jonasport is a breathing air warmer designed to help you stay active in cold weather.

The steel heat-exchanger grid captures the warmth of your exhaled breath and uses it to heat the inhaled air. At the same time, the grid humidifies the dry winter air, making it more comfortable for your respiratory system.

INSTRUCTIONS FOR USE

1. The Jonasport warmer stays securely in place with the Velcro strap. Position the split strap high enough around the crown of the head. The warmer should be carefully placed in front of the mouth and nose so that it does not cover the eyes.
2. Do not cover the steel mesh. If necessary, you can gently bend the mesh to fit the contours of your face.
3. Eyeglass wearers can slightly bend the metal clip on the top of the warmer to reduce fogging. If fogging occurs — for example, when driving or in other potentially dangerous situations — stop using the product immediately.
4. The product is intended only for warming inhaled air and **does not filter particles or pollutants**.
5. The product is for **personal use only**. After each use, rinse the warmer under running water. For more thorough cleaning, wash with a liquid detergent 2–3 times a week, depending on frequency of use.
Note: The soft edges of the warmer must not be exposed to temperatures above 60 °C.
6. The warmer is made of **polyethylene foam coated with soft fabric, acid-resistant stainless steel mesh**, and **elastic and Velcro straps**. Dispose of the product as **mixed waste**, and the metal parts as **metal waste**.